



BODE SCIENCE CENTER

Wir forschen für den Infektionsschutz.
www.bode-science-center.de



Manual reprocessing procedure for wipes dispensers with Bodedex® forte / Bacillol® AF

Discard residue of used fleece and liquid. Thoroughly rinse dispenser and lid with hot tap water.

Bodedex forte® and Bacillol® AF



Immerse all parts in a 1 % Bodedex forte solution (in a 30-litre disinfection bath or any other container that is large enough) for 10 minutes.



Take all parts out and mechanically clean them with a cloth soaked with 1 % Bodedex forte solution (from bath).



Thoroughly rinse dispenser and lid with cold water and completely dry all parts with a wipe.



Afterwards, wipe disinfect all surfaces with Bacillol AF (or any other surface disinfectant with a high alcohol content that dries without leaving any residue). Pay attention to the exposure time of at least 30 seconds. Make sure all surfaces are covered.



After all surfaces are completely dry, refill the dispenser with a new fleece roll and freshly prepared disinfection solution to be used for up to 28 days.